Calcium Chloride Solution (0.025M)

I. Intended Use

Pacific Hemostasis® Calcium Chloride Solution 0.025M (CaCl₂) is intended for use with particle based activators in performing the activated partial thromboplastin time (use with KONTACT®). Refer to product numbers 100308 and 100312 (KONTACT).

II. Reagent

For *in vitro* diagnostic use.

Composition: 0.2775% calcium chloride, 0.1% sodium azide and stabilizers.

Warning: Calcium Chloride Solution (0.025M) contains sodium azide. Sodium azide under acid conditions yields hydrazoic acid, an extremely toxic compound. Dilute with running water before discarding, and then flush with large volumes of water. These precautions are recommended to avoid deposits in metal piping in which explosive conditions may develop.

Store this product at 2-8°C.

Materials Provided: CaCl₂ (0.025M), 10 x 4 mL or 10 x 10 mL

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FISHER DIAGNOSTICS® LIMITED WARRANTY

Fisher Diagnostics (FD) warrants to the purchaser only that FD products will perform as described on their labeling and product literature. Purchaser must determine the suitability of FD products for their specific applications. FD's sole obligation will be, at its option, to either replace a non-conforming or defective product, or return the purchase price. FD DISCLAIMS ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTIES OF MERCHANTABILITY AND FITNESS FOR ANY PARTICULAR PURPOSE. Neither FD nor its affiliates shall, in any event, be liable for incidental or consequential loss or damage.

Pacific Hemostasis® is a registered trademark of Fisher Scientific Company L.L.C.
Fisher Diagnostics® is a registered trademark of Fisher Scientific Company L.L.C.

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Symbols Key

	Manufacturer
	In Vitro Diagnostic Medical Device
	Lot Number
	Use By
	Temperature Limitation
	CE Mark
	Catalogue Number
	Consult Instructions for Use
	Pack Date
	Authorized Representative in the European Community

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MDCI Ltd.