HDL Cholesterol Standard

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SYMBOLS IN PRODUCT LABELLING

EC REP	Authorised Representative		Temperature Limitation
IVD	For in vitro diagnostic use		Use by/Expiration Date
LOT	Batch code/Lot number		CAUTION. CONSULT INSTRUCTIONS FOR USE.
REF	Catalogue number		Manufactured by
	Consult instructions for use		

INTENDED USE

The Thermo HDL Cholesterol Standard is an aqueous solution of cholesterol to be used in the calibration of in vitro diagnostic methods for the determination of serum HDL cholesterol levels.

REAGENT COMPOSITION

Active Ingredients	Quantity	Cholesterol Concentration
Contents	30 mL	50 mg/dL (1.3 mmol/L)
1 bottle		

sodium azide, preservative, surfactant and isopropanol.

WARNING: Do not ingest. Avoid contact with skin and eyes. If spilt, thoroughly wash affected areas with water. Reagent contains Sodium Azide which may react with copper or lead plumbing. Flush with plenty of water when disposing. For further information consult the HDL Cholesterol Standard Material Safety Data Sheet. Exercise precaution when handling broken glass vials, as sharp edges can injure the user.

California state regulations require the following precaution for this product: **WARNING:** This product contains a chemical known to the State of California to cause cancer.

REAGENT PREPARATION

Thermo HDL Cholesterol Standard is supplied ready to use.

STABILITY AND STORAGE

Thermo HDL Cholesterol Standard is stable until the expiration date when stored at 2-30°C. Keep tightly capped to avoid evaporation.

DETERIORATION

The Standard should be a clear, colorless solution. Precipitation or cloudiness could indicate deterioration.

PROCEDURE

Use the Thermo HDL Cholesterol Standard as specified in the procedural insert for HDL cholesterol assay. Carefully follow the procedural insert in order to ensure accuracy.

ACCURACY

The Thermo HDL Cholesterol Standard has been referenced to NIST (NBS) Standard Reference Material.

LIMITATIONS

This standard is designed for use with the Thermo cholesterol tests. Performance may differ under other conditions or with other reagents.

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© 2008 Thermo Fisher Scientific Inc. All rights reserved.
a division of Fisher Scientific Company, LLC
a subsidiary of Thermo Fisher Scientific Inc.
EC REP MDCI Ltd.