# Shandon TBD-1™ and TBD-2™ Decalcifying Reagents

**Thermo**  
ELECTRON CORPORATION

Anatomical Pathology USA  
Clinical Diagnostics  
171 Industry Drive  
Pittsburgh, PA 15275, USA  
Tel: 1-800-547-7429  
+1 412 788 1133  
Fax: +1 412 788 1138  
[www.thermo.com/shandon](http://www.thermo.com/shandon)

Anatomical Pathology International  
Clinical Diagnostics  
93-96 Chadwick Road  
Astmoor, Runcorn  
Cheshire, WA7 1PR, UK  
Tel: +44 (0) 1928 566611  
Fax: +44 (0) 1928 565845  
[www.thermo.com/shandon](http://www.thermo.com/shandon)

## SHANDON DECALCIFYING REAGENTS

The decalcifying reagents, TBD-1 and TBD-2, were designed to give the technologist a choice of decalcification methods.

Most laboratories receive a wide variety of material to be decalcified. Because the decalcification time is different for each specimen, quality control is difficult in a busy laboratory. With TBD-1 and TBD-2 on hand, the technologist can now select the reagent to suit the circumstance. There is never a need to choose between quality and efficiency.

TBD-1 is a rapid decalcifying agent. TBD-1 will decalcify bone marrow biopsies in as little as 15 to 30 minutes. It will also decalcify thin pieces of dense bone in 3 to 12 hours without damage to soft tissues or loss of staining properties. TBD-1 is the reagent of choice for early results and fast diagnostic work.

TBD-2 is designed for safe, longer term decalcification. TBD-2 has a slower rate of reaction, which makes end-point determinations less critical.

Sections of bone have been left in TBD-2 for as long as 48 hours beyond the decalcification end-point without noticeable effect. TBD-2 is the reagent of choice for weekends, holidays, or when the laboratory workload does not permit close supervision of the decalcification procedure.

## WARNINGS AND PRECAUTIONS

### FOR IN VITRO DIAGNOSTIC USE.

### STORAGE

Store at room temperature. Keep containers closed when not in use. **DO NOT REUSE CONTAINER.**

---

---

## WARNINGS AND PRECAUTIONS

See Material Safety Data Sheets for warnings and precautions, as well as R & S code definitions. The following symbols are used on the product labels and packaging:

Definitions are at the rear of this booklet

---

---

## INSTRUCTIONS FOR USE

TBD-1 and TBD-2 are designed to be used full strength.

### A. Specimen Preparation

1. Bones and other calcified materials must be adequately fixed before placing them in TBD solutions. The fixative of choice is at the user's discretion.
2. Bone should be cut into sections of 3mm thickness or less.

### B. Procedure

1. Decalcification should be done in a glass or plastic container.  
**WARNING: DO NOT USE METAL CONTAINERS.**
2. Rinse excess fixative from specimen and place in TBD solution. Long washing after fixation or decalcification is not recommended.
3. The volume of TBD-1 or TBD-2 should be 20 times the volume of the specimen.
4. For more rapid decalcification, suspend the specimen in TBD-1 or TBD-2 and use a mechanical stirrer.  
**WARNING: DO NOT USE HEAT.**
5. Any of the routine methods for determining the end-point of decalcification can be used.
6. Decalcification in TBD-1 can take 15 minutes to 48 hours. Decalcification in TBD-2 can take several hours to several days.  
**NOTE:** Decalcification times vary depending on the size, thickness, density and type of specimen. More exact time limits must be based on the experience and judgement of the histotechnologist or pathologist.

### C. Additional Guidelines for TBD-1

1. Bone marrow biopsies will be decalcified in 15 to 30 minutes. They should be checked frequently to prevent over-decalcification.
2. Most bone sections larger than a bone marrow biopsy will decalcify within 3 to 24 hours. Caution must be used if the specimen will remain in TBD-1 overnight or on weekends and holidays.
3. If there is a possibility of over-decalcification:
  - a. Remove the specimen from TBD-1, rinse and return specimen to TBD-1 during work hours.
  - b. Use TBD-2.

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