Shandon Formal-Fixx™ Ready-to-Use
10% Neutral Buffered Formalin

Shandon Formal-Fixx™ Concentrate
Phosphate-Free

Thermo
ELECTRON CORPORATION

Anatomical Pathology USA
Clinical Diagnostics
171 Industry Drive
Pittsburgh, PA 15275, USA
Tel: 1-800-547-7429
+1 412 788 1133
Fax: +1 412 788 1138
www.thermo.com/shandon

Anatomical Pathology International
Clinical Diagnostics
93-96 Chadwick Road
Astmoor, Runcorn
Cheshire, WA7 1PR, UK
Tel: +44 (0) 1928 566611
Fax: +44 (0) 1928 565845
www.thermo.com/shandon

FORMAL-FIXX™

Formal-Fixx Ready-to-Use and Formal-Fixx Concentrate are neutral buffered formalins designed for use with light and electron microscopy. The phosphate-free buffering system in both prevents the formation of free formic acid, and its associate artifact, formalin pigment.

The osmolarity of these fixatives allows them to be used not only for light microscopy, but for electron microscopic studies as well. This eliminates the need to presort specimens for EM.

Formal-Fixx Ready-to-Use is a 10% neutral buffered formalin that is premixed and ready to use from the container. Formal-Fixx Concentrate is easily diluted to become a 10% neutral buffered formalin.

WARNINGS AND PRECAUTIONS FOR IN VITRO DIAGNOSTIC USE.

STORAGE:

Store at room temperature. Do not store with hydrochloric acid. Keep containers tightly closed when not in use.

WARNINGS AND PRECAUTIONS

See Material Safety Data Sheets for warnings and precautions, as well as R & S code definitions. The following symbols are used on the product labels and packaging:

Definitions are at the rear of this booklet.

INSTRUCTIONS FOR USE

1. Formal-Fixx Ready-to-Use is to be used full strength
2. Formal-Fixx Concentrate needs to be diluted. Mix 1 part of the concentrate with 4 parts distilled or deionized water. A one liter bottle of concentrate will make 5 liters of 10% neutral buffered formalin.
3. Use a routine histologic fixatives. For optimum results, the minimum tissue/fixative ratio should range from 1:15 to 1:20, and specimens should be placed into fixative immediately following removal from the patient for a period of time that will ensure adequate fixation.
4. Specimens may be stored in these solutions; they will not cause excessive shrinkage or hardening.

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