Shandon Consul-Mount™ Cytology Formulation

Thermo
ELECTRON CORPORATION

Anatomical Pathology USA
Clinical Diagnostics
171 Industry Drive
Pittsburgh, PA 15275, USA
Tel: 1-800-547-7429
+1 412 788 1133
Fax: +1 412 788 1138
www.thermo.com/shandon

Anatomical Pathology International
Clinical Diagnostics
93-96 Chadwick Road
Astmoor, Runcorn
Cheshire, WA7 1PR, UK
Tel: +44 (0) 1928 566611
Fax: +44 (0) 1928 565845
www.thermo.com/shandon

SHANDON CONSUL-MOUNT™, CYTOLOGY FORMULATION

CYTOLOGY FORMULATION CONSUL-MOUNT™ is a xylene-based mounting medium for use with glass coverslips. It is clear and colorless and will not discolor with age. The formula contains an antioxidant to inhibit stain fading. Its refractive index is 1.495 + 0.005 and is soluble in both xylene and toluene.

WARNINGS AND PRECAUTIONS

FOR IN VITRO DIAGNOSTIC USE

WARNING: FLAMMABLE

Store containers in areas approved for flammables. Do not handle or store near heat, sparks, flames, or strong oxidants.

DANGER: HARMFUL OR FATAL IF SWALLOWED. IF SWALLOWED, DO NOT INDUCE VOMITING. CALL A PHYSICIAN IMMEDIATELY. VAPOR HARMFUL. Reasonable care should be taken when using all laboratory reagents. Use with adequate ventilation. Keep containers closed when not in use.

WARNINGS AND PRECAUTIONS

See Material Safety Data Sheets for warnings and precautions, as well as R & S code definitions. The following symbols are used on the product labels and packaging:

Definitions are at the rear of this booklet

INSTRUCTIONS FOR USE:**Conventional Coverslipping**

Coverslip one slide at a time to prevent brown artifact from forming, and use the minimal amount of medium necessary to effectively bond the coverglass to the slide.

SUGGESTED PROTOCOL:

1. Lay the coverslip on an absorbent surface.
2. Place a drop or two of mounting medium toward the edge of the coverslip.
3. Remove a slide from clearing agent.
4. Quickly wipe the back of the slide dry.
5. Invert the slide so that the specimen side faces down.
6. Lower the slide until it reaches the coverslip, starting at one long edge. Gently press until the mounting medium spreads over the entire surface to be joined.
7. Turn over the slide/coverslip assembly.
8. If necessary, drain extra medium by standing the slide on its edge on an absorbent towel.
9. Gently press out any noticeable air bubbles.

Note: Thermo has not validated the protocol illustrated here and takes no responsibility for its use. Customers should always validate protocols before placing any reliance on them.

Automated Coverslipping

Follow directions for use with your automated coverslipping instrument.

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