Thermo

ELECTRON CORPORATION

INSTRUCTIONS FOR USE

SHANDON CYTOCLIP™ SLIDE CLIP TPX SAMPLE CHAMBERS FILTER CARDS

For use with Shandon Cytospin® cytocentrifuge

A78310253

Versie 3

Anatomical Pathology

93 - 96 Chadwick Road, Astmoor, Runcorn, Cheshire, WA7 1PR, UK

Tel: +44 (0) 1928 566611;

Fax: +44 (0) 1928 565845;

www.thermo.com/shandon

171 Industry Drive, Pittsburgh, Pennsylvania, PA 15275, U.S.A.

Tel: +1 412 788 1133;


Fax: +1 412 788 1138;

www.thermo.com/shandon

SHANDON CYTOSPIN® ACCESSORIES INSTRUCTIONS FOR USE

Shandon Cytoclip, TPX chambers, filter cards

For use with Shandon Cytospin® cytocentrifuge

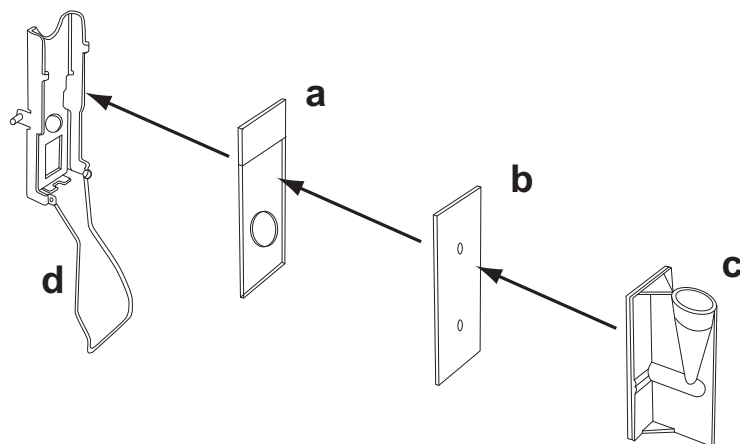To load the Shandon Cytoclip slide clip with a re-useable sample chamber and filter card:

- 1 Fit the glass slide (a)
- 2 Fit the filter card (b)
- 3 Fit the re-useable sample chamber (c)
- 4 Pull up the spring (d) and press it into the two retaining hooks to hold the chamber in place.
- 5 Make sure the sample chamber and filter card are correctly aligned

Note: To make sure that the sample does not touch the slide before centrifugation, always load the sample after the sample chamber has been loaded into the Sealed Head

For full instructions describing the use of the Shandon Cytospin, how to load and balance the Sealed Head, and how to load the sample, please refer to the Shandon Cytospin Operator Guide.

To remove the slide from the Cytoclip, hold the slide clip firmly and push the spring against the slide to release it from the hooks. Move the spring away from the hooks and allow the sample chamber to be released from the clip. Remove the slide.

Store the Shandon Cytoclips, TPX Chambers and Filter Cards in a clean, dry environment when not in use.

MAKE SURE THAT GOOD LABORATORY PRACTICE IS FOLLOWED WHEN POTENTIALLY HAZARDOUS MATERIALS ARE HANDLED

FILTER CARDS ARE SINGLE USE ONLY - Do not use more than once. Safely dispose of the filter cards immediately after use.

Cleaning and decontamination instructions are detailed in the following table.




Thermo Electron Corporation makes every endeavour to ensure that the information contained in its support documentation is correct and clearly stated but does not accept responsibility for any errors or omissions. The development of Thermo products and services is continuous. Make sure that any published information that you use for reference is up to date and relates to the status of the product. If necessary, check with Thermo or your local Thermo representative.

All information contained in this manual is proprietary and confidential, and the exclusive property of Thermo Electron Corporation. This manual is protected by copyright and any reproduction is prohibited. This manual is for use only by the individuals to which it has been made available by Thermo Electron Corporation.

© 2002-2004 Thermo Electron Corporation. All rights reserved.

This manual may not, in whole or in part, be copied, photocopied, reproduced, translated, or converted to any electronic or machine readable form without prior written consent of Thermo

Table 1. Cleaning and Decontamination

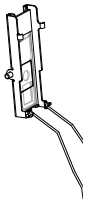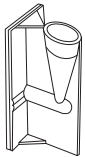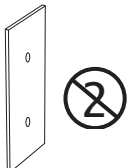Part	Frequency	Instructions
 Cytoclip Slide Clip	Daily	<p>Check: Check that the clip is not damaged</p> <p>Decontaminate: Autoclave at 121°C (250°F) for 15 minutes Alternatively, disinfect by submerging in a 10% solution of commercial bleach in water for not less than one hour</p> <p>Clean: Rinse in clear water and hot air dry before use</p> <p>Note: For Slide Clip (part number 5991052M), snap open the spring; for spring loaded Slide Clip (part number 5991017M), turn the pressure release screw fully counter clockwise to release the spring pressure</p>
 Reusable Sample Chamber	After use	<p>Check: Check that the chamber is not damaged</p> <p>Decontaminate: Autoclave at 121°C (250°F) for 15 minutes. DO NOT autoclave the plastic cap Note: Discolouration due to repeated autoclaving does not adversely affect the performance of the Sample Chamber Alternatively, disinfect by submerging in a 10% solution of commercial bleach in water for not less than one hour</p> <p>Clean: Use a cotton tipped applicator swab and hot soapy water to clean the outlet port and funnel. Rinse in clean water, shake to remove excess water, then dry DO NOT use a wire brush</p>
 Filter Cards	After use	<p>Decontaminate: Discard into a phenolic disinfectant, or Autoclave at 121°C (250°F) for 15 minutes. Alternatively, submerge in a 10% solution of commercial bleach in water for not less than one hour</p> <p>Dispose of in the approved manner according to local procedures depending on the specimens and materials used</p>

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