Shandon EZ Single and Double Cytofunnel® Sample Chambers Instructions for use

(Note - the numbered steps in this document refer to label instruction illustrations)

For use with Shandon Cytospin® cytocentrifuge

1 To Load the EZ Cytofunnel: Fit the glass slide into the slide support. Make sure the slide is correctly oriented i.e. with the frosted label end to the top and the frosted side towards the EZ Cytofunnel. The slide should be located behind the slide stops and against the back of the slide support.

2 Close the EZ Cytofunnel.

Note

1 Disposable EZ Cytofunnels have filter cards permanently attached to simplify loading and unloading the slide and are designed to be used only once.

3 To load the Shandon Cytospin Sealed Head, remove the lid and place up to 12 assembled EZ Cytofunnels into the slots. Make sure the assembled EZ Cytofunnels are correctly fitted into the Sealed Head and that they remain tilted forward.

Note

1 When an EZ Cytofunnel is placed into the Slide Plate in the Sealed Head, the chamber is kept at a suitable angle to keep the sample away from the slide. When the Shandon Cytospin is started, the EZ Cytofunnel swings into an upright position that allows the sample to be spun onto the slide.

4 To load the samples, make sure that the EZ Cytofunnel assembly is in the rest (tilted) position and use a pipette to carefully load the EZ Cytofunnel with between 0.1ml and 0.5ml (max.) of cells in suspension.

Note

1 Do not fill the EZ Cytofunnels with more than the amounts specified below:
EZ Single Cytofunnel (white filter card) 0.5ml maximum
EZ Single Cytofunnel (brown filter card) 0.4ml maximum
EZ Double Cytofunnel 0.25ml maximum in each side

5 Fit the cap on each EZ Cytofunnel to avoid spillage.

6 Make sure that the EZ Cytofunnels are evenly distributed in the Sealed Head, to ensure the Sealed Head is balanced. For example, place EZ Cytofunnels into positions 1, 5 and 9 if only three EZ Cytofunnels are available, or positions 1, 4, 7 and 10 if only 4 EZ Cytofunnels are available.

7 Replace the lid of the Sealed Head and push down the centre button to lock.

8 Place the Sealed Head into the Shandon Cytospin.

9 Start the appropriate program.

DO NOT RUN FOR MORE THAN 60 MINUTES

- 10 When the Shandon Cytospin has stopped spinning, remove the Sealed Head to a safe and convenient area. Make sure the Sealed Head is kept level.
- 11 Remove the lid of the Sealed Head. (Pull the centre button to unlock it).
- 12 Remove each EZ Cytofunnel assembly in turn from the Sealed Head.
- 13 To remove the slide from the EZ Cytofunnel, press the thumb pad release lever to open the slide support.
- 14 Remove each slide by lifting the slide up and away from the EZ Cytofunnel to avoid smearing the cells.
- 15 Discard the Disposable Cytofunnel directly into the appropriate waste receptacle depending on the specimens and materials used, according to your local laboratory procedure.
- 16 Allow the liquid that remains on the slide to evaporate quickly. Place the slide in 95% alcohol. Stain the slide as required.

Note

1 The slide can be spray fixed before staining.

SINGLE USE ONLY - Do not use the EZ Single or Double Cytofunnel more than once.

Store unused EZ Cytofunnels in a clean, dry environment before use.

MAKE SURE THAT GOOD LABORATORY PRACTICE IS FOLLOWED WHEN POTENTIALLY HAZARDOUS MATERIALS ARE HANDLED

For full instructions describing the use of the Shandon Cytospin, please refer to the Operator Guide.

Clinical Diagnostics
Anatomical Pathology
171 Industry Drive
Pittsburgh
PA 15275, USA
Tel: 1-800-547-7429
+1 412 788 1133
Fax: +1 412 788 1138
www.thermo.com/shandon

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