

Thermo Scientific
Richard-Allan
Scientific
Disposable Base
Molds

**Instructions
For Use**

Disposable Base Molds

Thermo Scientific Richard-Allan Scientific disposable base molds are designed for use with all standard size tissue cassettes and embedding rings. They are molded from rigid plastic and the block design provides greater base support and smaller cutting surfaces. Cleaning associated with metal molds is not required and the transparency facilitates tissue orientation.

Instructions for Use

1. Fill the base mold with the desired embedding medium.
2. Orientate the tissue in the embedding medium and cool until the embedding medium begins to solidify.
3. Place the tissue cassette on the base mold and fill to the top with embedding medium.
4. Cool until the block is easily removed from the base mold.
5. Dispose of the base mold.

Warning and Precautions

See Material Safety Data Sheets for warnings and precautions, as well as R & S code definitions.

See container labels for warnings and precautions.

Order Information

Product	Size	Quantity	REF
Disposable Base Mold 7 x 7mm	125/box	4 boxes/cs.	58949
Disposable Base Mold 15 x 15mm	125/box	4 boxes/cs.	58950
Disposable Base Mold 24 x 24mm	125/box	4 boxes/cs.	58941
Disposable Base Mold 30 x 24mm	125/box	4 boxes/cs.	58952
Disposable Base Mold 37 x 24mm	125/box	4 boxes/cs.	58953

©2008 Thermo Fisher Scientific Inc. All rights reserved. All trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

Anatomical Pathology



Richard-Allan Scientific

Subsidiary of Thermo Fisher Scientific
4481 Campus Drive, Kalamazoo, MI 49008
Phone: 1 (800) 522-7270
Phone: + 1 (269) 544-5600
www.thermo.com/pathology



Microm International GmbH

Subsidiary of Thermo Fisher Scientific
Robert-Bosch-Str. 49
D-69190 Walldorf/Germany
Phone: + 49(0)6227-8360



IS8132 R4/08