

Thermo Scientific  
Richard-Allan  
Scientific  
Base Mold Release

**Instructions  
For Use**

## Base Mold Release

Thermo Scientific Richard-Allan Scientific Base Mold Release is designed to be used at full strength and should not require any additives. It is used to coat base molds for easy separation of paraffin blocks from the mold and reduce paraffin build up in the mold. This product is environmentally safe and is available in a 4-ounce, non-aerosol pump spray bottle.

### Instructions For Use

#### Procedure

1. Clean and completely dry base molds are sprayed with an even coat of Base Mold Release.
2. The base molds are now ready to use.
3. Additional application of Base Mold Release can be applied as needed through out the day.

### Warnings and Precautions

See Material Safety Data Sheets for warnings and precautions, as well as R & S code definitions.

See container labels for warnings and precautions.

## Order Information

Product	Size	Quantity	REF
Base Mold Release	4oz. (118mL)	12/cs.	6600

©2008 Thermo Fisher Scientific Inc. All rights reserved. All trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

## Anatomical Pathology



### **Richard-Allan Scientific**

Subsidiary of Thermo Fisher Scientific  
4481 Campus Drive, Kalamazoo, MI 49008  
Phone: 1 (800) 522-7270  
Phone: + 1 (269) 544-5600  
[www.thermo.com/pathology](http://www.thermo.com/pathology)

EC REP

### **Microm International GmbH**

Subsidiary of Thermo Fisher Scientific  
Robert-Bosch-Str. 49  
D-69190 Walldorf/Germany  
Phone: + 49(0)6227-8360



IS8104 R6/08